



Gentex Europe

3-5 Commerce Road
Stranraer
Scotland
DG9 7DX
United Kingdom
Phone: +44 (0) 1776 704421
Fax: +44 (0) 1776 706342

Appendix 1:

Protocol for cleaning of PureFlo 3000 Hood – Powered Air Respirator

Always consult the manufacturer's instructions for decontaminating equipment.

1. To Clean Head top:

Wipe down with a soft cloth dampened with neutral detergent solution. Rinse with clean water. Do not soak hoods during cleaning.

2. Disinfect Head top:

Wipe head top with a clean soft cloth dampened with a disinfectant solution (e.g. a 0.1% chlorine solution or a solution containing 1000 ppm available chlorine). Alternatively, a combined detergent/disinfectant may be used. Do not soak hoods.

3. Rinse head Top:

Wipe all components cleaned with a clean soft cloth dampened with clean warm water. Note: It is important to rinse off chlorine-based products. While rinsing is preferred, certain other disinfectants may not require this step. Follow the user instructions for the product selected." Ensure all headgear components are allowed to completely air dry prior to reuse or storage.